

Notes: _____

Provox StabiliBase

The Provox StabiliBase has a convex shape and a firm base with vertical stabilizing bars, combined with a FlexiDerm type of adhesive. It also has a 3-section peel-off liner to put the adhesive on step by step. The convex shape is designed to better fit, especially for users with deep stoma's and the firm base with stabilizing bars supports the neck during (handsfree) speech. Anyone who tolerates a FlexiDerm type of adhesive can try Provox StabiliBase to see if it is beneficial for them.

This page shows you step by step how to put the Provox StabiliBase adhesive on. You can remove your old adhesive and prepare your skin as you normally do.



Putting on Provox StabiliBase – Step by Step

1. Put the adhesive upside-down on a flat surface
2. Hold the outer liners down with two fingers
3. Peel away the middle liner
4. Apply the middle part over the stoma and massage onto the skin
5. Peel off one side and massage onto the skin
6. Repeat on the other side
7. Massage the adhesive onto the skin
8. Give the adhesive time to set by not speaking immediately. Wait for 20 to 30 minutes.

Wearing the Provox StabiliBase

The Provox StabiliBase may stay on for as long as it provides an airtight seal for speech. You can help the seal by using low effort to speak and keeping the area around the stoma clean of mucus.

TIP: If you like, it is also still possible to peel the liner of completely and put the adhesive on all at once, or to start from one of the sides and work step by step from left to right or right to left.

