

Justification:	Function:	Electronic signature justification:	Signed: Date (yyyy-mm-dd) - Time (hh:mm):
Issued:	DD	Pontus Eklund - X-PONEKL	2020-05-25 - 14:00
Reviewed:	DD	Jon Berg - JONBER	2020-05-25 - 14:46
Approved:	DD	Fredrik Calais - FRECAL	2020-05-25 - 15:53
Released:	DD	Pontus Eklund - X-PONEKL	2020-05-26 - 08:11

This document has been electronically signed by the persons above.

Released

Product Information

Provox® FreeHands FlexiVoice



Product description:

The Provox FreeHands FlexiVoice consists of two parts assembled together, a speaking valve for single patient use and a disposable HME cassette.

The speaking valve is made of plastic and the membrane is made of silicone. The HME cassette is also made of plastic and a salt treated polyurethane foam.

The speaking valve has two modes; Automatic Speaking Mode and Locked Mode. Rotating the top of the speaking valve moves the device into the automatic speaking or the locked position. Speaking can be done both by using the automatic speaking valve and by manual occlusion of the opening in the front. Manual occlusion is possible in both modes.

Product Information

Document ID:	PF058-01-TechInfo	Edition:	03
Manufacturer:	Atos Medical AB Kraftgatan 8, P.O. Box 183, SE-242 22 Hörby, Sweden		
Classification: (EU) 2017/745	Class I, Rule 1		
Intended Use:	The Provox FreeHands FlexiVoice combines pulmonary rehabilitation using a Heat and Moisture Exchanger with voice rehabilitation using an Automatic Speaking Valve or Manual Occlusion, in laryngectomized patients using a voice prosthesis.		
CE Mark:	Yes, the devices are CE marked.		
GMDN code:	36071		
Sterilization:	Non-sterile		
Raw material:	Speaking valve: PP, Silicone and POM. Arch: MABS		
Latex information:	Not manufactured with natural rubber latex		
Biological origin:	The device is not manufactured with materials derived from human or animal source.		
Handling and storage:	Store the product dry and away from sunlight at room temperature. Excursions permitted between 2°C - 42°C.		
Waste handling and disposal:	Waste handling and disposal should be carried out in agreement with medical practice and applicable national laws and legislations. Used product may be a potential biohazard.		
Hazardous components:	None		
Expiration date:	3 years after manufacturing.		
Packaging:	<p>Ref 7757: Box with 6 pcs [plastic bag with 5 pcs Provox FreeHands HME Cassette] + 3 plastic jars with 1 pc Speaking Valve of each version (light, medium, strong) + plastic bag with 1 pc Removal Aid + plastic bag with 2 pc Arch + 1 pc IFU Ref 10740.</p> <p>Ref 7760: Box with 3 plastic jars with 1 pc Speaking Valve of each version (light, medium, strong) + plastic bag with 1 pc Removal Aid + plastic bag with 2 pc Arch + 1 pc IFU Ref 10740.</p> <p>Ref 8161: Box with 1 plastic jars with 1 pc Speaking Valve Provox FreeHands FlexiVoice Light + plastic bag with 1 pc Removal Aid + plastic bag with 1 pc Arch + 1 pc IFU Ref 10740.</p> <p>Ref 8162: Box with 1 plastic jars with 1 pc Speaking Valve Provox FreeHands FlexiVoice Medium + plastic bag with 1 pc Removal Aid + plastic bag with 1 pc Arch + 1 pc IFU Ref 10740.</p>		

Product Information

Ref 8163:

Box with 1 plastic jars with 1 pc Speaking Valve Provox FreeHands FlexiVoice Strong + plastic bag with 1 pc Removal Aid + plastic bag with 1 pc Arch + 1 pc IFU Ref 10740.

Ref 8165:

Box with 5 plastic bag with 1 pc Arch in each bag.

Ref 8166:

Box with 1 plastic jars with 1 pc Speaking Valve Provox FreeHands FlexiVoice XtraStrong + plastic bag with 1 pc Removal Aid + plastic bag with 1 pc Arch + 1 pc IFU Ref 10740.

Devices under Basic UDI-DI: 7331791-HME-0-000-0007-XW

REF	Name	UDI-DI
7757	Provox FreeHands FlexiVoice Set Plus	07331791008276
7760	Provox FreeHands FlexiVoice Set	07331791008283
8161	Provox FreeHands FlexiVoice Light	07331791008290
8162	Provox FreeHands FlexiVoice Medium	07331791008306
8163	Provox FreeHands FlexiVoice Strong	07331791008313
8165	Provox FreeHands FlexiVoice Arch (5 pcs)	07331791010408
8166	Provox FreeHands FlexiVoice XtraStrong	07331791010668
7757-18	Provox FreeHands FlexiVoice Set Plus	07331791013508
7760-18	Provox FreeHands FlexiVoice Set	07331791013515
8161-18	Provox FreeHands FlexiVoice Light	07331791013461
8162-18	Provox FreeHands FlexiVoice Medium	07331791012365
8163-18	Provox FreeHands FlexiVoice Strong	07331791013485
8166-18	Provox FreeHands FlexiVoice XtraStrong	07331791013492



Atos Medical AB compatible products:

Range	BASIC UDI-DI
Provox FreeHands HME	7331791-HME-0-000-0003-XJ

Justification:	Function:	Electronic signature justification:	Signed: Date (yyyy-mm-dd) - Time (hh:mm):
Issued:	QA	Peter Sundsten - X-PETSUN	2020-04-22 - 10:12
Reviewed:	DD	Jon Berg - JONBER	2020-04-22 - 15:00
Approved:	DD	Fredrik Calais - FRECAL	2020-04-23 - 09:11
Released:	QA	Peter Sundsten - X-PETSUN	2020-05-13 - 14:27

This document has been electronically signed by the persons above.

Released

Product Information

Provox® FreeHands HME



Product description:

Provox FreeHands HMEs are designed to allow automatic tracheostoma closure (instead of digital stoma occlusion) for voice prosthesis users (e.g. Provox, Voice Prosthesis), while providing heat-and-moisture-exchange (HME, via the Provox FreeHands Cassette) for pulmonary rehabilitation. The device should be adjusted for the end user by a clinician trained in voice and pulmonary rehabilitation, e.g. a speech pathologist, before it can be used without supervision – *this does not apply for Provox FreeHands FlexiVoice.*

Product Information

Document ID:	PF022-02-TechInfo	Edition:	07
Manufacturer:	Atos Medical AB Kraftgatan 8, P.O. Box 183, SE-242 22 Hörby, Sweden		
Classification: (EU) 2017/745	Class I, Rule 1		
Intended Use:	Provox FreeHands HME Cassette/Moist/Flow is intended for single use for spontaneously breathing laryngectomized patients, utilizing a voice prosthesis and must be used in combination with a Provox FreeHands speaking valve, a Provox cap or Digitop O2. Provox FreeHands combines pulmonary rehabilitation with its Heat and Moisture Exchanging functionality with voice rehabilitation using an Automatic Speaking Valve or Manual Occlusion. The HME conditions inhaled air by retaining heat and moisture from the exhaled air. The device also partially restores lost breathing resistance.		
CE Mark:	Yes, the devices are CE marked.		
GMDN code:	58705 (Tracheostoma protective filter)		
Sterilization:	Non-sterile		
Raw material:	Cassette: Styrene-ethylene-butadiene-styrene (SEBS) and Polyurethane (PUR). Foam: Polyurethane (PUR) with calcium chloride (CaCl ₂)		
Latex information:	Not manufactured with natural rubber latex		
Biological origin:	The device is not manufactured with materials derived from human or animal source.		
Handling and storage:	Store the product dry and away from sunlight at room temperature. Excursions permitted between 2°C - 42°C.		
Waste handling and disposal:	Waste handling and disposal should be carried out in agreement with medical practice and applicable national laws and legislations. Used product may be a potential biohazard.		
Hazardous components:	None		
Expiration date:	3 years after manufacturing.		
Packaging:	HMEs are packed in plastic bags made of polyethylene that are placed in a cardboard box. The following configurations are present:		

REF	#HMEs per plastic bag	#Plastic bags in cardboard box	#HMEs in product
8220, 8221	10	3	30
8220-18, 8221-18	10	3	30
8222-18, 8223-18	10	2	20

Product Information

Devices under Basic UDI-DI: 7331791-HME-0-000-0003-XJ

REF	Name	UDI-DI
8220	Provox FreeHands HME Moist (30 pcs)	07331791008368
8221	Provox FreeHands HME Flow (30 pcs)	07331791008375
8220-18	Provox FreeHands HME Moist (30 pcs)	07331791012372
8221-18	Provox FreeHands HME Flow (30 pcs)	07331791013553
8222-18	Provox FreeHands HME Moist (20 pcs)	07331791013560
8223-18	Provox FreeHands HME Flow (20 pcs)	07331791013577

Atos Medical AB compatible products:

Range	BASIC UDI-DI
Provox StabiliBase	7331791-ADH-0-000-0000-CQ
Provox XtraBase	7331791-ADH-0-000-0000-CQ
Provox StabiliBase OptiDerm	7331791-ADH-0-000-0000-CQ
Provox Flexiderm	7331791-ADH-0-000-0000-CQ
Provox Optiderm	7331791-ADH-0-000-0000-CQ
Provox LaryTube	7331791-LTU-0-000-0002-3E
Provox LaryButton	7331791-LTU-0-000-0000-38
Provox HME Cassette Adaptor	7331791-HME-A-000-0003-F5
Provox HME Cap	7331791-HME-A-000-0002-F2
Provox FreeHands FlexiVoice	7331791-HME-0-000-0003-XJ

Justification:	Function:	Electronic signature justification:	Signed: Date (yyyy-mm-dd) - Time (hh:mm):
Issued:	DD	Pontus Eklund - X-PONEKL	2020-04-20 - 15:59
Reviewed:	DD	Jon Berg - JONBER	2020-04-20 - 16:55
Approved:	DD	Fredrik Calais - FRECAL	2020-04-21 - 14:55
Released:	DD	Pontus Eklund - X-PONEKL	2020-10-28 - 16:38

This document has been electronically signed by the persons above.

Released

Provox® FreeHands Support™ Adhesive**Product description:**

The Provox FreeHands Support system consists of two parts; Provox FreeHands Support and Provox FreeHands Support Adhesive.

The reusable Provox FreeHands Support is available in three different variants; 'Flat', 'Medium' and 'Deep'. The three variants, as well as the possibility to adjust the metal ring and plastic parts, enable personal fit. Provox FreeHands Support Adhesive is a single-use adhesive that is attached to the skin, beneath the Provox Adhesive.

Product Information

Document ID:	PF078-02-TechInfo	Edition:	05
Manufacturer:	Atos Medical AB Kraftgatan 8, P.O. Box 183, SE-242 22 Hörby, Sweden		
Classification: (EU) 2017/745	Class I, Rule 1		
Intended Use:	Provox FreeHands Support provides support to the Provox Adhesive when using a Provox hands-free speaking valve after total laryngectomy. The device is a single patient use device with a single use adhesive.		
CE Mark:	Yes, the devices are CE marked.		
GMDN code:	62175 (Stomal appliance skin-adherent patch, non-sterile)		
Sterilization:	Non-sterile		
Raw material:	Provox FreeHands Support Adhesive consists of a plastic plate (Polycarbonate, PC) and an adhesive tape (Acrylate and Polyester film).		
Latex information:	Not manufactured with natural rubber latex		
Biological origin:	The device is not manufactured with materials derived from human or animal source.		
Handling and storage:	Store the product dry and away from sunlight at room temperature. Excursions permitted between 2°C - 42°C.		
Waste handling and disposal:	Waste handling and disposal should be carried out in agreement with medical practice and applicable national laws and legislations. Used product may be a potential biohazard.		
Hazardous components:	None		
Expiration date:	3 years after manufacturing.		
Packaging:	Provox FreeHands Support Adhesive is packed in a blister package made of PET film and with a re-sealable top film (PP). It is then packed in a cardboard box.		

Product Information

Devices under Basic UDI-DI: 7331791-HME-A-000-0004-F8

REF	Name	UDI-DI
8024	Provox FreeHands Support Adhesive (15pc)	07331791009303

Atos Medical AB compatible products:

Range	BASIC UDI-DI
Provox FreeHands Support	7331791-HME-A-000-0000-EU


Justification:	Function:	Electronic signature justification:	Signed: Date (yyyy-mm-dd) - Time (hh:mm):
Issued:	DD	Pontus Eklund - X-PONEKL	2020-04-20 - 15:59
Reviewed:	DD	Jon Berg - JONBER	2020-04-20 - 16:56
Approved:	DD	Fredrik Calais - FRECAL	2020-04-21 - 14:56
Released:	DD	Pontus Eklund - X-PONEKL	2020-10-28 - 16:38

This document has been electronically signed by the persons above.

Released

Product Information

Provox® FreeHands Support™

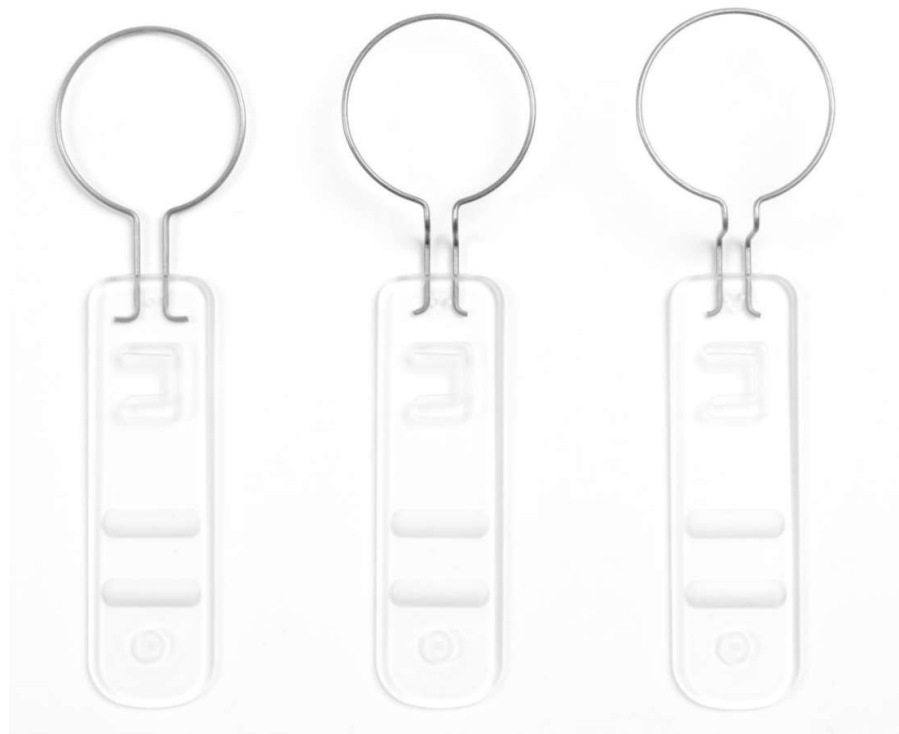

Product description:

The Provox FreeHands Support system consists of two parts; Provox FreeHands Support and Provox FreeHands Support Adhesive.

The reusable Provox FreeHands Support is available in three different variants; 'Flat', 'Medium' and 'Deep'. The three variants, as well as the possibility to adjust the metal ring and plastic parts, enable personal fit. Provox FreeHands Support Adhesive is a single-use adhesive that is attached to the skin, beneath the Provox Adhesive.

Product Information

Document ID:	PF078-01-TechInfo	Edition:	04
Manufacturer:	Atos Medical AB Kraftgatan 8, P.O. Box 183, SE-242 22 Hörby, Sweden		
Classification: (EU) 2017/745	Class I, Rule 1		
Intended Use:	Provox FreeHands Support provides support to the Provox Adhesive when using a Provox hands-free speaking valve after total laryngectomy. The device is a single patient use device with a single use adhesive.		
CE Mark:	Yes, the devices are CE marked.		
GMDN code:	62155 (Tracheostomy base plate, reusable)		
Sterilization:	Non-sterile.		
Raw material:	Provox FreeHands Support consists of a ring of stainless steel and a plastic part (Polycarbonate, PC).		
Latex information:	Not manufactured with natural rubber latex		
Biological origin:	The device is not manufactured with materials derived from human or animal source.		
Handling and storage:	Store the product dry and away from sunlight at room temperature. Excursions permitted between 2°C - 42°C.		
Waste handling and disposal:	Waste handling and disposal should be carried out in agreement with medical practice and applicable national laws and legislations. Used product may be a potential biohazard.		
Hazardous components:	None		
Expiration date:	3 years after manufacturing.		
Packaging:	Provox FreeHands Support is packed in a cardboard box together with instructions for use.		

Product Information

Devices under Basic UDI-DI: 7331791-HME-A-000-0000-EU

REF	Name	UDI-DI
8020	Provox FreeHands Support Starter Set	07331791009266
8021	Provox FreeHands Support Flat	07331791009273
8022	Provox FreeHands Support Medium	07331791009280

Atos Medical AB compatible products:

Range	BASIC UDI-DI
Provox FreeHands Support Adhesive	7331791-HME-A-000-0004-F8
Provox FreeHands FlexiVoice	7331791-HME-0-000-0007-XW

Justification:	Function:	Electronic signature justification:	Signed: Date (yyyy-mm-dd) - Time (hh:mm):
Issued:	DD	Pontus Eklund - X-PONEKL	2020-05-27 - 07:50
Reviewed:	DD	Jon Berg - JONBER	2020-05-27 - 08:13
Approved:	DD	Fredrik Calais - FRECAL	2020-05-27 - 08:43
Released:	DD	Jon Berg - JONBER	2020-06-23 - 14:53

This document has been electronically signed by the persons above.

Released

Product Information

Provox® HME Cap

**Product description:**

Provox HME Cap is a dome-shaped titanium ring for rehabilitation after total laryngectomy. It allows use of the Provox FreeHands HME Cassettes without the Provox FreeHands HME Speech Valve. The front opening of the cap can be occluded manually to speak.

Product Information

Document ID:	PF019-01-TechInfo	Edition:	05
Manufacturer:	Atos Medical AB Kraftgatan 8, P.O. Box 183, SE-242 22 Hörby, Sweden		
Classification: (EU) 2017/745	Class I (1.1 Rule)		
Intended Use:	<p>Provox HME Cap is a single patient use, dome-shaped titanium ring, that allows use of Provox FreeHands HME cassette (REF 8220, 8221) without Provox FreeHands FlexiVoice.</p> <p>Provox HME Cap is only intended for use when using Provox FreeHands FlexiVoice is not recommended, i.e. when sleeping.</p> <p>Provox HME Cap cannot be used with any other type of HME cassette. The front opening of the cap can be occluded manually to speak. Provox HME Cap can be cleaned and reused.</p>		
CE Mark:	Yes, the devices are CE marked.		
GMDN code:	58705 (Tracheostoma protective filter)		
Sterilization:	Non-Sterile		
Raw material:	Titanium		
Latex information:	Not manufactured with natural rubber latex		
Biological origin:	The device is not manufactured with materials derived from human or animal source.		
Handling and storage:	Store the product dry and away from sunlight at room temperature. Excursions permitted between 2°C - 42°C.		
Waste handling and disposal:	Waste handling and disposal should be carried out in agreement with medical practice and applicable national laws and legislations. Used product may be a potential biohazard.		
Hazardous components:	None		
Expiration date:	5 years after manufacturing.		
Packaging:	HME Cap is separately packed in a mini grip plastic bag, then a cardboard box.		

Product Information

Devices under Basic UDI-DI: 7331791-HME-A-000-0002-F2

REF	Name	UDI-DI
7730	Provox HME Cap	07331791003011

Atos Medical AB compatible products:

Range	BASIC UDI-DI
Provox FreeHands HME Cassette	7331791-HME-0-000-0003-XJ